

SARS-CoV-2 Ag Rapid Test For Self-testing

RTG0101 / RTG0104 / RTG0102 / RTG0103

1 Test / 2 Tests / 5 Tests / 20 Tests

Autobio Diagnostics Co., Ltd.

offers more than 400 in vitro diagnostic products including CLIA(microplate based CLIA and magnetic particle based CLIA),ELISA,POCT(Point of Care Test),Microbiology and Biochemistry. As an ISO9001 and EN ISO13485 manufacturer, Autobio supplies high quality products through its well established sales network and is renowned as a reliable partner. For details please visit en.autobio.com.cn. Autobio Diagnostics Co., Ltd. | NO.87 Jingbei Yi Rd | National Eco & Tech Zone | Zheng zhou City | China 450016.

You can find us on:

Facebook: @AutobioGlobal

Youtube Channel: Autobio Diagnostics

Email: info@autobio-diagnostics.com

WhatsApp: +8618595767835

Tel: +86-371-6798-5313



SARS-CoV-2 Ag Rapid Test

CE 1434

- is a colloidal gold method for **Self-Testing At Home** as an aid in the diagnosis of SARS-CoV-2 infections.
- is a rapid, qualitative determination of SARS-CoV-2 nucleocapsid protein antigen in **Anterior Nasal Swabs Specimen**.
- takes **4 Steps** to give you a reliable result.
- indicates your health status in **15 Minutes**.



1 Set, 4 Steps, 15 Minutes, 100% Comfortable & Reliable


1 Set

- 1 set contains a bottle of Extraction Reagent, an individual packaging Cassette, a disposable Sterile Swab, a Garbage Bag and an IFU.
- 1 set is enough to tell you the health status.




4 Steps

- 4 steps provide you a reliable result – a gentle anterior nasal swab is self-collected, then the swab is swirled in the Extraction Reagent. Drip four drops of samples into the cassette. Start the timer and read your result after 15 minutes.
- Before you start the test, you should carefully read and follow the INSTRUCTIONS FOR USE. Wrong steps may lead to a false result.

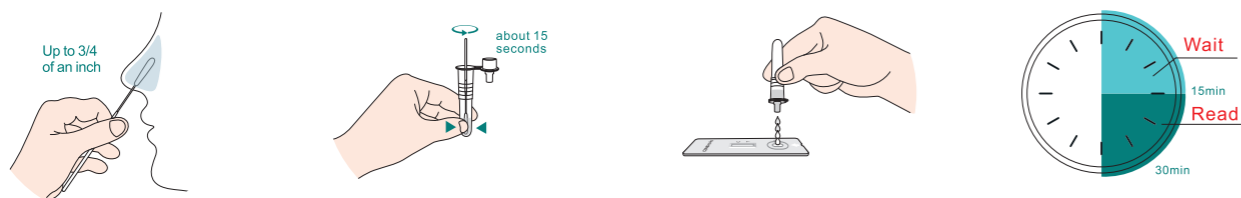

15 Minutes

Your result will be available after 15 minutes. Please share your test results with your healthcare provider and take corresponding measures considering different results – Positive, Negative or Invalid.

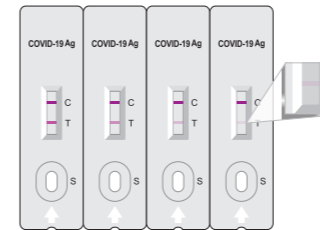

Positive Results:

The test is Positive if:

- There are two colored-lines.
- There are two colored-lines, even if the color of the line next to the T is Faint. Look Closely!



Negative Results:

The test is Negative if:

- There is a colored-line next to the C and NO line next to the T.
- * In case of suspicion, repeat the test after 1-2 days because the coronavirus cannot be accurately detected in all phases of an infection.



Invalid Results:

The test is not working and should be repeated if:

- No lines are present.
- There is only a colored-line next to the T.
- * You will need to conduct another test!

100% Comfortable & Reliable

- Anterior nasal swab is available instead of uncomfortable nasopharyngeal specimen.
- Compact package for self-testing at home.
- Rapid results to show your health status when you go back to work, go back to school or public gathering.
- Easy visual results without instrument requirement.
- Room temperature storage.
- Clinical sensitivity: 94.55% (95% CI, 88.51% - 97.97%).
- Clinical specificity: 100.00% (95% CI, 99.19% - 100.00%).



FAST RESULT
Results In 15 Minutes



EASY TESTING
A Simple Nasal Swab



HIGHLY ACCURATE



Scan the QR code for more product information